## TECHNIKER INFO

### *Transferpapiere*

### *Art. 201000 ff. und 300710 ff.*

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*ref. 201000 following and 300710 following*

#### **Product description:**

With paper transfers it is possible to color vacuum forming Orthoses or Protheses sockets.

#### **Intended use:**

Paper transfers are used exclusively to color PP- or PE sheets.

#### **Use instructions:**

**1.**  
Cut your paper transfers according to your PP or PE sheet. The paper transfers should overlap the edges of the sheets (for easy removing).

**2.**  
Clean your sheet free of grease e.g. with Acetone.

**3.**  
Heat the cut sheet in the oven.

**4.**  
Put the paper transfers on the heated sheet as soon as the sheet becomes transparent. Rub the paper transfers with a soft cloth thoroughly on the sheet to get a good result.

**5.**  
React the color 45 sec. on the sheet before you remove the paper transfers.  
**Caution:** Only remove the paper on one edge of the sheet to check if the intensity is enough. If you prefer a deeper color you can react the color a few more seconds on the sheet.

#### **Tips for vacuum forming of models:**

- Use silicon spray if you form the model with hands to get a nice surface.
- If your sheet is thinner than 3 mm, heat the sheet one minute more after removing the paper transfers.
- Take care, that the temperature isn't too high. At high temperatures the sheet becomes adhesive and removing paper transfers will become more difficult.

This documentation is only for information and is legally not binding. For special applications, there should be done little tests in advance.